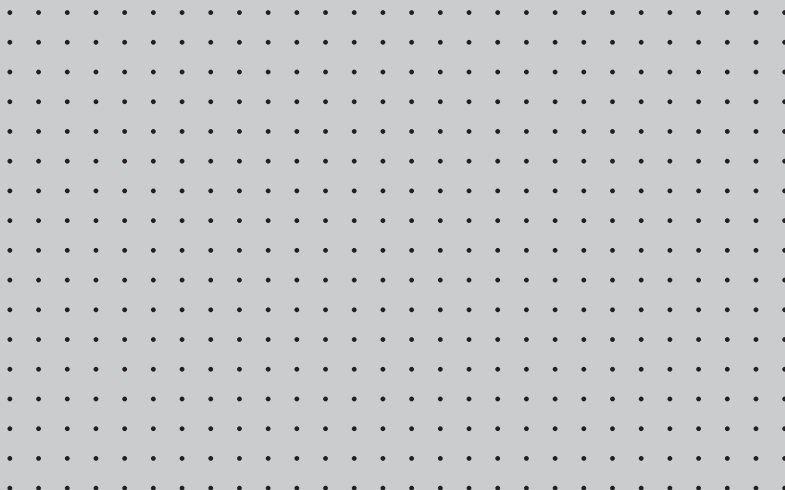
GOMARCO

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NUCOL ADULT MEDICAL MATTRESS

OPERATING HEALTH LICENSE Nº 9002-PS

INSTRUCTIONS FOR USE AND OPERATION:
IT IS IMPORTANT TO KEEP FOR FUTURE REFERENCE.
PLEASE READ CAREFULLY.

ÍNDICE

1. INDEX	2
1.1. THE COMPANY	3
1.2. LEGEND OF SYMBOLS	4
2. MEDICAL MATTRESSES RANGE	5
2.1. MODELS	5
2.2. TECHNOLOGIES/MATERIAL FEATURES	6 - 8
2.3. CERTIFICATIONS	9
3. USAGE AND MAINTENANCE INFORMATION	10
3.1. SUPPLY CONDITIONS	10
3.2. MEDICAL PURPOSE	10
3.3. USAGE CONDITIONS	10
3.4. ACCESSORIES	11
3.5. MAINTENANCE AND CLEANING	12
3.6. OPTIMAL DURATION	12
4. WARNINGS AND CONTRAINDICATIONS	12-13
5. PRODUCT DISPOSAL	14
6. WARRANTY	14-15
7. MEDICAL DEVICE CERTIFICATE	16
8. EU DECLARATION OF CONFORMITY	17

1 • INTRODUCTION

1.1. The Company

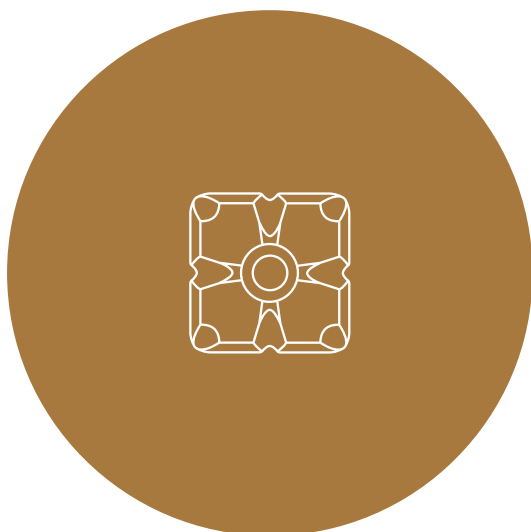
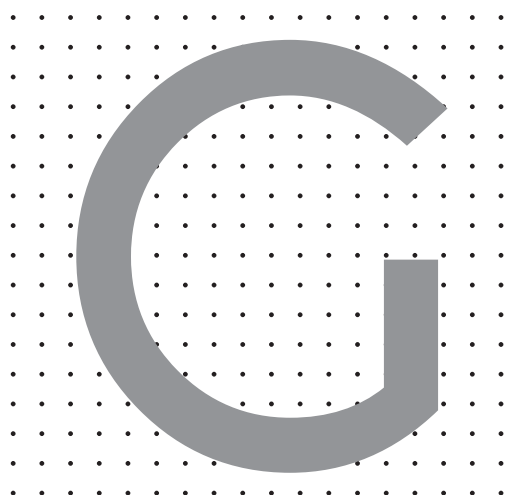
Gomarco's journey began over 40 years ago, after significant human and professional efforts.

We design and manufacture the sleep experience you deserve, and the one you want for your loved ones. All our work is focused on improving the quality of rest and leveraging the latest advances to offer you new sensations that enhance your life and dreams.

The dedication and passion of a whole team committed to improving sleep and innovating through quality and the development of new materials and technologies have allowed us to offer new experiences in the world of rest.

The evolution of sleep is in our DNA. Contributing to a healthy life and improving people's comfort is our driving force and the primary goal of Gomarco.

Thanks to these values, Gomarco now offers you the mattress you have just purchased, a product carefully designed and engineered to care for your health and enhance the quality of your sleep.

COLCHONES GOMARCO S.L.
CIF/ B-05.517.511 Ctra. Villena KM 3.5, P.O. Box 403 Yecla-Murcia-España
(+34) 968 79 00 90 info@gomarco.com

1 • INTRODUCTION

1.2. Legend of Symbols

Iconography of the various elements and their meaning, found in documents and labels used to identify and track the numerous products offered by our brand.

Safety indications regarding potential hazards of accidents or injuries to the user

Serial number

Reference to an external document

Medical device Class 1

CE marking

Product reference

UDI unique identification code

Manufacturer

Date of manufacture

2 • MEDICAL MATTRESSES RANGE

2.1. Model

The ADAPT SAC mattress is composed of a combination of a topper, a core, and a base. This combination is designed to address different user conditions, adapting to their anatomical and biological characteristics. The bases provide a sense of firmness and adaptability to the whole system. They include a layer of springs that enhance air circulation and breathability throughout the mattress.

The core consists of viscoelastic springs of varying densities, combined to offer different pressure zones. The topper enhances the core's adaptability and firmness, improving the thermal comfort perceived by the user.

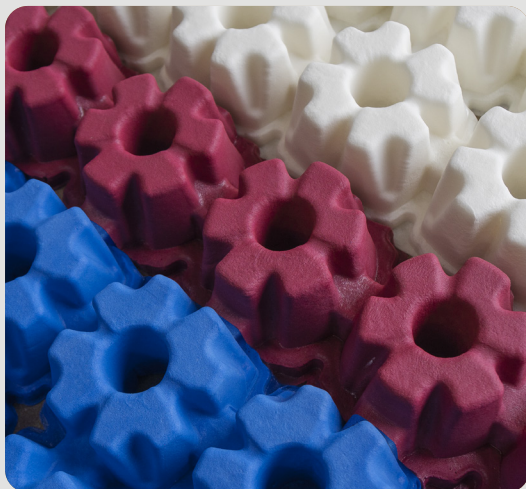
2.2. Technologies/Material Features

A breathable mattress allows air to flow freely through its interior, preventing the buildup of moisture, heat, and dust mites, thereby reducing the risk of allergies, irritations, and excessive sweating. These factors will not affect the quality of sleep or development. Breathability helps the mattress adapt to temperature changes, maintaining a pleasant and consistent thermal comfort throughout the night. Additionally, it contributes to the hygiene and durability of the product, as it prevents the formation of bad odors, stains, and mold. A clean and well-maintained mattress lasts longer and provides better performance.

The Revertop® system, an invention by GOMARCO (PATENTED SYSTEM. Utility model No.: E-1.236.304 U), offers the possibility of easily changing sensations with a simple rotation. You can switch from the summer side to the winter side without having to flip the mattress over. The Revertop system allows quick access to the interior of the mattress, making it easy to ventilate, which extends the mattress's lifespan even further. Our topper is the perfect combination of the best materials on the market, designed for greater comfort. With the Revertop® system, you'll enjoy two different sensations in one mattress. Its unique reversible zipper system ensures that the topper is fully integrated into the mattress.

NUCOL® 

**PATENTED
SYSTEM**
· ES2632255 ·

The Nucol® System is made up of high-quality viscoelastic components that, through different levels of firmness and distribution throughout the mattress, adapt to the shape of our body. Currently, there are three different firmness levels available for configuring a Nucol mattress (firm, medium, and soft). Additionally, each Nucol unit has channels that allow air to circulate, refreshing the air inside the mattress and making it more breathable and cooler, thereby preventing excessive heat buildup. In short, the Nucol® System is an innovative, unique support, perfect for the most demanding sleep, as it combines the best of springs—firmness and breathability—with the best of viscoelastic foam—adaptability and comfort.

SPRINGSAC®
SOFT

Springsac is a pocket spring core that provides motion isolation, prevents disturbances from movements, and ensures total adaptability and high breathability.

2.2. Technologies/Material Features

The new ViscoTitanium®Gel formula from GOMARCO consists of high-density gel particles and titanium particles. We combine the best characteristics of both components: cooling and enhanced energy management for the body. It provides cooling, optimizes rest, and accelerates muscle recovery.

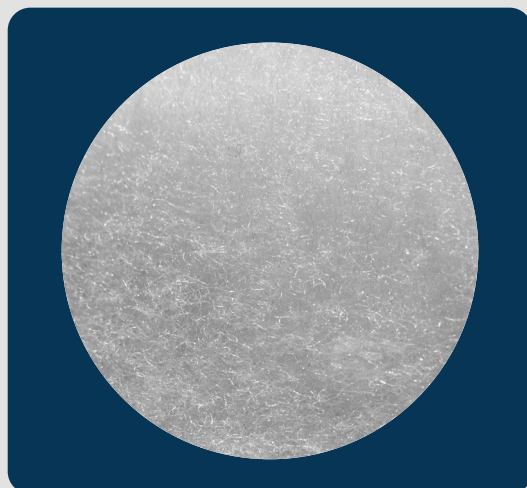
- **Better heat dissipation.**
- **Boosts your energy.**

**REINFORCED
FRAME**

**TEX
REFORCED**

Extra-firm perimeter frame: When sitting on the edge of the bed, it prevents the springs from shifting to the sides, and it is perforated for breathability.

STRETCH^H

Technical fabrics are designed to enhance air circulation from the core. The fabrics in contact with the skin are crafted to relieve the sensation of heat during warmer months and provide a cozy feeling during colder months.

2 · MEDICAL MATTRESSES RANGE

2.3. Certifications

The fabrics used in the manufacturing of Gomarco mattresses are certified with STANDARD 100 by OEKO-TEX®, a product certification that ensures the absence of harmful substances in any item that is part of the textile chain, such as fiber, yarn, fabrics, finished garments, metal accessories, foams, and mattresses.

ISO 9001:2015 QUALITY MANAGEMENT SYSTEM

The ISO 9001 standard is a set of requirements that ensures a company meets international quality standards in its products and services. For a mattress manufacturer, it means having a quality management system that guarantees customer satisfaction, compliance with applicable regulations, continuous process improvement, and the management of risks associated with its activities.

GOMARCO SLEEP CHAIR PROJECT

In collaboration with the University of Murcia, Gomarco has created an innovative project in the sleep sector: the Gomarco Sleep Chair. With a clear goal in mind—to improve our clients' sleep and rest—we have designed four action plans focusing on dissemination, awareness, research, and training.

3 • USAGE AND MAINTENANCE INFORMATION

3.1. Supply Conditions

This mattress is sold open. If the customer requests it rolled up, it will be at their own responsibility. In such a case, the mattress should be opened immediately upon receipt and allowed to return to its original state on a firm, flat surface. Do not store the mattress rolled up for more than 2 days after receiving it. The recovery period should be between 24-48 hours. Wrinkles in the fabric are considered normal.

3.2. Medical Purpose

Designed to prevent and relieve conditions that disrupt restorative and healthy sleep, such as back pain, lumbago, or any muscle pain under normal conditions. Medical mattresses are designed to improve sleep quality in adults and reduce insomnia. They also enhance sleep quality for people with thermal regulation issues (such as hyperhidrosis or excessive sweating).

The technical fabrics used in the topper allow adaptation for both cold and warm months, aiding in optimal thermoregulation, which helps shorten the time to fall asleep and improves overall sleep quality. For those suffering from hyperhidrosis, the combination of "cool effect" fabrics and enhanced ventilation will increase the feeling of dryness on the surface.

Anti-Bedsore Function. The development of pressure ulcers is mainly due to a combination of pressure concentration and high humidity. The cores and toppers of the medical mattress offer better pressure distribution and improved ventilation conditions, helping to prevent the occurrence of bedsores.

3.3. Usage and Maintenance Recommendations

The effectiveness of mattresses depends on how they are used. We recommend the following:

- Do not jump or stand on the mattress..
- Do not bend or force the mattress, including the corners or sides.
- Keep it clean and protected from moisture and external agents with a breathable cover.
- Regularly ventilate the room.
- Do not expose the mattress to direct sunlight or other sources of light and/or heat.
- Use a firm and breathable base of the same size.
- Avoid storing the mattress upright for prolonged periods.
- Always transport it using its original packaging.
- We recommend rotating the mattress from head to foot every **4 - 6 months**.
- The optimal temperature range for ensuring proper functionality is between +10°C and +40°C. Outside of this range, its properties may be diminished.
- Pay attention to the weight guidelines on the label. Exceeding these limits may reduce the mattress's performance and cause premature wear.
- It is not suitable for weights under 25kg, as the materials used would have no effect.
- Frequent room ventilation improves the mattress's breathability and aids in the quick evaporation of any retained moisture (removing the topper multiplies this effect)).
- Do not use electric blankets, as they may compromise the durability of the mattress and reduce its effectiveness.

3 • USAGE AND MAINTENANCE INFORMATION

3.4. Accessories

Accessories, such as bedding, that are to be used with Gomarco medical mattresses must be suitable, as they may affect the medical function.

Check with your bed manufacturer to ensure that the mattress can be used with it without compromising the mattress's properties.

If your bed has lifting mechanisms (both manual and mechanical), consider the weight of the mattress and its accessories. We recommend a perforated base, avoiding a slatted base to improve the mattress's breathability.

Try to choose bedding accessories that are appropriate for the climatic conditions, as this can negatively impact service conditions. Select breathable bedding accessories.

Consult your supplier for recommended accessories (bedding, accessories, etc.).

3.5. Maintenance and Cleaning

To properly maintain your mattress:

- Follow the usage and maintenance rules outlined on the label attached to the mattress and in this manual.
- Conduct visual inspections of the overall condition of the mattress, as the presence of defects (such as holes, open seams, or faulty zippers) can negatively impact performance by allowing moisture to enter the core and cause damage.
- The mattress must be cleaned and disinfected using a common disinfectant designed for this purpose, ensuring it does not contain harmful and/or corrosive substances that could damage it.
- Periodically ventilate and rotate the mattress, especially in the first few months, to enhance initial adaptation and improve air circulation and moisture evacuation.
- Do not use heating devices (such as irons or hairdryers) to dry the mattress's fabrics, as this may compromise the durability of the materials used in its construction.

3 • USAGE AND MAINTENANCE INFORMATION

3.6. Optimal Duration

Mattresses are designed with the best materials to ensure a long lifespan.

The mattresses and the materials used in their construction have undergone the necessary tests to guarantee adequate strength and quality.

4 • WARNINGS AND CONTRAINDICATIONS

The product is contraindicated in cases of:

Overweight: The product's effectiveness may be completely diminished due to excessive tension on the mattress surface.

Spinal Injuries: Particularly if the spinal cord is affected, and in conditions where strong fixation of the back to a hard surface is required.

Presence of Skeletal Cervical Traction: In this case, there is a significant risk of worsening the patient's condition.

Consult your doctor if you have any doubts.

4 • WARNINGS AND CONTRAINDICATIONS

The product is contraindicated in cases of:

- Maintenance operations other than those described in the "Maintenance" section should not be performed on the mattress.
- Do not allow the mattress to be handled by unauthorized third parties. If necessary, contact the manufacturer exclusively.
- For safety reasons, do not smoke in bed..
- DO NOT spill liquids on the mattress surface.
- Always seek assistance when changing the mattress position.
- Do not jump or stand on the mattress.
- Do not bend or force the mattress, including the corners or sides.
- Keep it clean and protected from moisture and external agents with a breathable cover.
- Regularly ventilate the room.
- Do not expose it to sunlight or other sources of light and/or heat.
- Use a firm and breathable base of the same size.
- Avoid storing it upright for prolonged periods.
- Always transport using its original packaging.
- After a short period of use, you may notice some indentation marks; this is due to the fillings adapting.
- Damage to the mattress resulting from failure to follow these usage instructions is not the manufacturer's responsibility.

5 • PRODUCT DISPOSAL

This product must NOT be disposed of with household waste. Please take it to the designated waste collection point as determined by local authorities.

Protect the environment and recycle this product at the nearest waste collection facility when it reaches the end of its useful life. If you need information on how to proceed, please contact your local municipality, and they will provide the necessary details.

Waste disposal must comply with legal regulations regarding waste treatment in each country

6 • WARRANTY

COLCHONES GOMARCO, S.L., guarantees the owner of this product free repair or replacement for manufacturing defects within the legally established time frames, effective at the time of purchase.

According to Royal Legislative Decree 1/2007 of the LGDCU, the following situations do not entitle you to warranty service:

- Deterioration caused by improper or inadequate use of the product.
- Wear and tear resulting from the use of the product, including stains and tears in fabrics and/or handles.

COLCHONES GOMARCO, S.L., reserves the right to change, without prior notice, the padding, fabrics, composition, etc., of any of its products as long as it aims to improve that product. The prices in this catalog include transportation to the designated store or warehouse. For special delivery or assembly requests, please contact your sales representative.

All our merchandise deliveries are accompanied by the corresponding invoices subject to the established tax, VAT. The signed delivery slip is proof of the merchandise received correctly. Any claims for missing items or damage will be the customer's responsibility, unless noted on the delivery slip at the time of unloading.

All our products come with a warranty label (if absent, the warranty will be according to law) that outlines their recommended use. Failure to comply with these recommendations voids the warranty for the purchased product. The customer must inspect the product for any issues as applicable. To make a warranty claim effective, the customer must notify their sales representative with the following information: Invoice number, invoice date, a photograph of the invoice; model name and size; reason for the issue.

In the event of a manufacturing defect, payment will be processed once the merchandise is received and inspected at our facilities. Any return to the factory must be done in a plastic cover to prevent damage. Otherwise, Gomarco Mattresses, S.L. will charge the customer for any damage incurred during transport. In the case of a dispute between the parties, the customer waives any personal jurisdiction they may have and expressly submits to the courts and tribunals of Yecla.

6 • WARRANTY

The buyer must inspect the merchandise upon delivery and note any obvious discrepancies on the delivery slip at that moment. Additionally, the buyer has 7 days to claim any visible defects, such as stains, scuffs, seams coming undone, and tears in zippers. After this period, it will be understood that the merchandise has been received in conformity, and no claims for these defects can be made afterward.

To file a claim, you must present the original invoice or receipt as proof of purchase at the establishment where you acquired the product. Any returns to the factory must be made with a plastic cover and properly packaged to avoid further deterioration.

Regarding incidents, our specialized agents will assist in assessing whether it is a manufacturing defect or misuse by the customer. Subsequently, our quality department will verify the issue with the product and determine the appropriate action in each case.

The following situations are not included in our warranty:

- Dirt or stains on the mattress.
- Defects due to improper storage.
- Odors.
- Moisture due to insufficient ventilation.
- Defects caused by repairs or handling performed by third parties.
- Use of chemical products for cleaning.
- Defects caused by using the mattress without adequate support.
- Non-compliance with usage and maintenance recommendations.
- Wrinkles in padded fabrics or bands when sitting are considered normal.
- Variations in dimensional characteristics within the tolerances set by European standard
UN-EN 1334:1996 +/-20mm in width and +/-20mm in length.

7 • MEDICAL DEVICE CERTIFICATE

One of the most innovative and beneficial systems for sleep is the NUCOL® system. It consists of a core made of injected viscoelastic foam, with vertical and horizontal shapes and channels that allow for air circulation and mattress breathability without sacrificing adaptability or firmness. This design prevents the accumulation of moisture, dust mites, and bacteria, creating a fresh and healthy environment.

However, the NUCOL system not only provides comfort and softness to the mattress, but it also has significant preventive and therapeutic functions. It enhances sleep quality for individuals with thermal regulation issues, such as those suffering from excessive sweating (hyperhidrosis), fever, or allergies. By allowing air to flow freely through the mattress, it regulates body temperature and prevents sudden changes that can disrupt sleep.

Another advantage of our NUCOL® technology is that it supports anti-bedsore functions, reducing pressure points between the body and the mattress, which improves blood circulation. This results in multiple health benefits, as it helps prevent numbness in the extremities, cramps, or localized pain in the back and neck.

The NUCOL® system is composed of high-quality viscoelastic components that allow for different levels of firmness and distribution throughout the mattress, adapting to the shape of our body. These health issues, combined with daily stress, can lead to insomnia; therefore, the NUCOL® system will significantly help prevent it.

THERMOREGULATOR

**EXTRA
BREATHABLE**

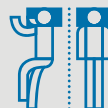
ERGONOMIC

**REINFORCED
LUMBAR AREA**

ANTIBACTERIAL

**UNDISTURBED
SLEEP**

MADE IN SPAIN

**MOTION
ISOLATION**

CLASE 1

NUCOL ADULT MEDICAL MATTRESS
OPERATING HEALTH LICENSE N° 9002-PS

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